Ultimate Comfort

Modell: Smart

C20-0094
2302

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Dear user!

Our congratulations on the choice of your new Ritter product and many thanks for your confidence! This modern device was produced by our development- and production engineers with great carefulness.

You and your team will gain absolute joy and benefit of your new Ritter device, if you and your employees, after proper mounting on site, follow the technical and application-oriented advices from the handing over demonstration.

Despite of a careful introduction into the operation, smaller or greater questions will arise in the course of time. This operating manual shall give you answers to these questions. To make day-to-day work easy for you and to be sure, that all technical or legal regulations and the maintenance notes receive their attention, we recommend a careful examination of this manual for you and your responsible employees too.

If you face technical problems, our service hotline
49 (0) 7351 / 52 925-37 is available for you.

And now a lot of joy with your Ritter equipment.

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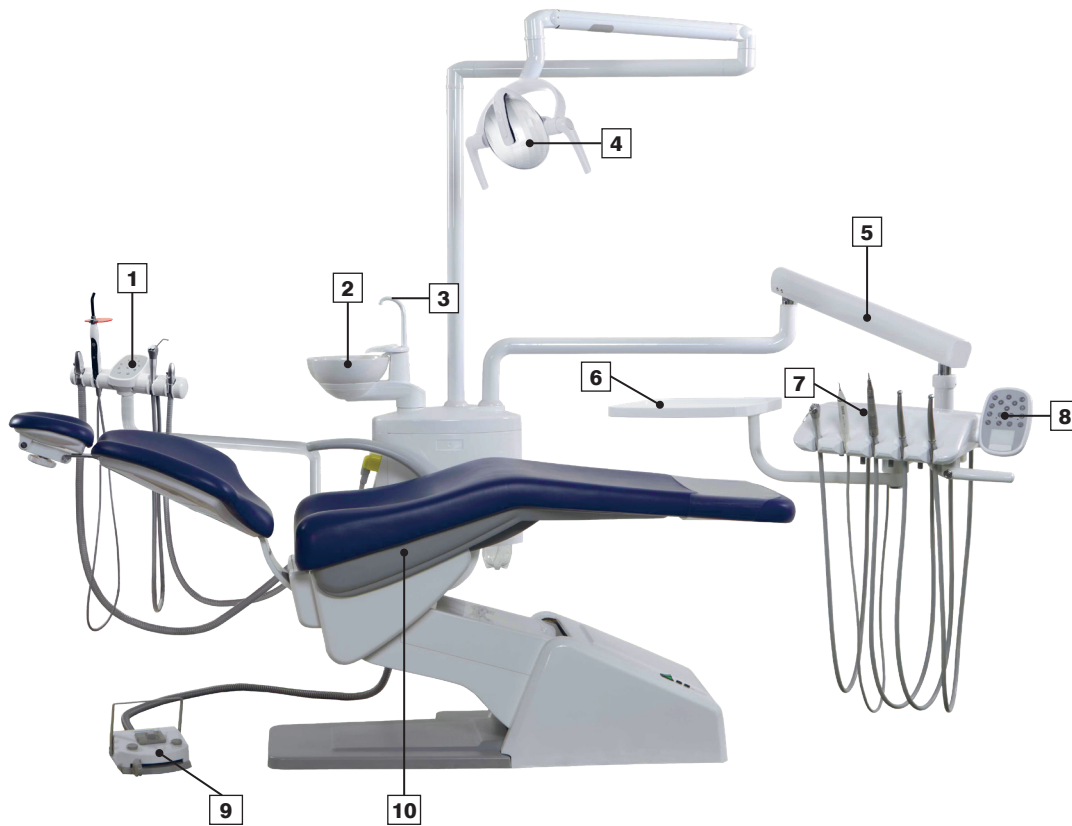
Date of issue: 2023-10-18
Original instruction for use

Version: C20-0094-2302

Explanation:

C20 = Code for user manual
0094 = Document identification
2302 = Version number consisting of:
(20)23 current year and
02 current revision of document

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No.	Description
1	Assistant element
2	Spittoon bowl
3	Cup filler
4	Operating lamp
5	Table arm
6	Tray
7	Dentist element
8	Dentist keyboard
9	Multifunctional foot controller
10	Patient chair

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General

Foreword.....	3
Overview.....	4
Table of content.....	5
Technical data.....	6
Operating/transport and storage conditions.....	7
accessories and detachable parts.....	8
Signs and symbols and name plate.....	10
General informations.....	13
General guidelines.....	15
Safety information.....	17
Working area.....	18
Notes.....	19

Dentist element

Device description.....	22
Features.....	23
Control elements.....	24
Foot controller.....	25
Arm brake.....	26
Turbine/Air motor/Micromotor.....	27
Regulation of spray-water.....	28
Air pressure regulation, Prompt tone.....	29
Ultrasonic scaler.....	30
Syringe.....	31
Operating lamp „CoolLite“.....	32
Operating lamp „SunLite“ (optional).....	34
Operating lamp „Standard Light“ (optional).....	36

Assisntant element

Operation assistant element.....	40
Spittoon bowl rinsing/Cup filler/Heating.....	42
Suction/Aqua bottle system.....	43
Aqua bottle system.....	44

Patient chair

Working area.....	48
Manual control.....	49
Program control.....	50
Headrest.....	52

Maintenance

Daily.....	56
Maintenance.....	59
Cleaning.....	60
Troubleshooting.....	66
Self test, Factory settings.....	71

Appendix

Setting up drawing.....	76
Air-water-schematic.....	77
Wiring diagram.....	78
Safety inspection.....	80
EMC declaration.....	81
Terms of Guarantee.....	83

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Dental work place

Type:	Ultimate Comfort
Model:	Smart
Rated voltage:	230 V ± 10% 110 V ± 10%
Frequency:	50/60 Hz ± 10 Hz
Rated input:	900 VA
Fuse:	10 A time lag
Protection class:	I
Type of applied part:	B
Heater:	24 V AC, 120 VA
Degree of water proof:	Non-water-tight
Mode of operation:	DAB*
Weight:	255 kg
Dimensions:	190x85x180 cm (l x w x h)
Rotation range of the spittoon:	≥180°
Air source:	
Air pressure:	5.5 bar...8 bar
Air flow:	≥50 l/min
Water supply:	
Water pressure (input):	2.0...4.0 bar**
Water flow:	≥10 l/min
Requirements for water quality:	
Optimum of water hardness:	1.42 ... 1.78 mmol/l (8 ... 10°dH)

Water unit

Rotational angle of spittoon bowl: 180°

Patient chair

Mode of operation:	DAB* (2 min/18 min)
(Intermittent Duty Time Rating:	On: 2 min Off: 18 min)
Maximum height (seat cushion ⇄ floor):	800 mm
Minimum height (seat cushion ⇄ floor):	400 mm
Maximum stroke:	400 mm
Backward angle of backrest:	102°-192°
Up and down sliding stroke of head rest:	200 mm
Maximum loads:	135 kg

Dentist element:

vertical motion range
of the instrument table: 480 mm

Dental operating light „CoolLite“

Type:	LED
Rated voltage:	17-24 V AC 50/60 Hz 22-35 V DC
Rated input:	9 VA
Mode of operation:	DAB***
Protection class:	II
Illuminance:	3000-35000 lx (in distance of 700 mm)
Colour temperature:	5000 K
Light spot:	85 mm x 170 mm
Lifetime cycle:	5 years

Dental operating light „SunLite“ (optional)

Type:	LED
Rated voltage:	17-24 V AC 50/60 Hz
Rated input:	20 VA
Mode of operation:	DAB***
Protection class:	II
Illuminance:	8000-50000 lx (in distance of 700 mm)
Colour temperature:	4200-6000 K
Colour rendering index (CRI):	>85
Lifetime cycle:	50000 h

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Dental operating light „Standard Light LED“ (optional)

Type:	LED
Rated voltage:	12-17 V AC 50/60 Hz
Rated input:	6 VA
Mode of operation:	DAB***
Protection class:	II
Illuminance	8000-30000 lx (in distance of 700 mm)
Colour temperature:	5000 K
Light spot:	80 mm x 150 mm
Lifetime cycle:	5 years

Technical modifications reserved!

DAB* = continuous operation with intermittent load.
The permissible load times are adapted to the working mode of the dentist (please refer also to the operating manual for the individual units and/or instruments).

**=At site there must be a back-flush protective filter (mesh size 80-120 µm) for the water inlet pipe to the supply unit. The complete installation must be according to the official standard EN 1717 conducted.

DAB*** = continuous operation

Operating conditions

Temperature:	+10°C to +40°C
rel. humidity:	30% to 75%
Barometric pressure:	700 hPa to 1060 hPa

Transport and storage conditions

(original packing)

Temperature:	-20°C to +55°C
rel. humidity:	max 93%
Barometric pressure:	500 hPa to 1060 hPa

Micromotor

„Infinity LED“	
Type of applied part:	B

Ultrasonic Scaler

„Kolibri S/E“	
Type of applied part:	BF
Continuous working mode	

Curing light

„OPM LED“	
Type of applied part	B

Water unit

Part	Description
Spittoon bowl	Interchangeable for cleaning and sterilisation
Cup filler	
Cup holder	
Spittoon rinser	
Front cover of water unit	Removable for access to technical parts inside the water unit
Water bottle	Removable for refilling and cleaning purposes
Water unit linking arm cover	Removable to change positions of water unit, e.g. left-handed or right-handed operation of dental unit
Suction filter	Easily accessible for cleaning and changing of the filters
Operating lamp handles	The handles can be easily exchanged for cleaning

Dentist element

Part	Description
Turbine handpiece	Interchangeable for cleaning and sterilisation
Micromotor Infinity LED	
Scaler Kolibri S/E LED	
Polylight LED	
Syringe Luzzani 3F	
Oil catcher beneath dentist element	Removable for maintenance
Silicon covers	Interchangeable for cleaning and sterilisation
Stainless steel trays	

Assistant element

Part	Description
Big and small suction hoses including handpieces	Interchangeable for cleaning and sterilisation

Patient chair

Part	Description
Headrest	Removable for cleaning
Transparent foot cover	Interchangeable for cleaning

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Explanations to name plates

Icon	Explanation
	Serial number
	<u>CE-Approval</u> according to Directive 93/42/EEC for medical products
	The symbol of the crossed waste container on wheels indicates, that Ritter products are electrical equipment according to Directive 2012/19/EU. Within the EU-Countries these products have to be separately disposed. Ritter wants to encourage you, to minimize possible effects of waste to environment and health outside of the EU, too. Please follow the local directives for disposal of waste and make use of the separate collection of electrical equipment.
	There are specific warnings or precautions related to this product.
	Type B applied part
	Following the instruction for use
	Manufacturer/Date of manufacturing (according to ISO 15223)

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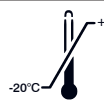
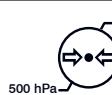
Icons on packaging

Icon	Explanations to type plates
	Fragile, Handle with care
	This way up
	Do not stack
	Keep dry
	Permissible temperature range
	Permissible range of humidity
	Permissible range of barometric pressure
	<u>CE-Approval</u> according to directive 93/42/EEC for medical products
	Serial number

Icons in this Instruction for use

Icon	Legend (used icons in this manual)
	Note
	Attention Observe special precautions or warnings for this product.

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Intended purpose

This dental treatment unit is used for diagnosis, therapy and dental treatment in humans. It positions the patient, illuminates the oral cavity and provides the drive for the dental instruments.

Indication

Not further specified.

Contraindication

Patients who cannot be positioned on the chair and cannot maintain the position required during treatment.

Patient target group

There are no specific restrictions based on age, weight, health status or ethnicity as long as patients are able to follow the user's instructions. The decision as to whether a patient is eligible for treatment with the product rests solely with the intended user. The maximum load of the patient chair must be observed.

Intended user

Trained and instructed dental professionals.

Complication

- Risk of crushing: When moving the backrest, the patient's arms must not be behind the back or moving dentist or assistant element
- Reversible glare if the LED treatment light is handled incorrectly.
- Risk of infection due to transmission of pathogens through cross infection
- Risk of infection due to transmission of pathogens
- Risk of overloading the dentist element, assistant element or patient chair

Clinical benefit

Both patient and user benefit from the ergonomically arranged components of the patient chair, dentist element, work field light, water unit and assistant unit.

The patient is positioned by the patient chair so that access to the oral cavity is possible for the user. The patient chair is adjusted in height and the inclination of the backrest is set accordingly.

Dental treatment is performed with additional instruments and devices. These are supplied with electrical voltage, compressed air, vacuum and water by the dental treatment unit.

Serious incidents

All serious incidents related to the product must be reported to the manufacturer and the competent authority of the Member State where the user is established.

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Our equipment complies with the requirements of medical products (MPDG) and Directive 93/42/EEC for Medical Devices, revised by EU-Directive 2007/47/EG from 2010-03-21.

It complies with the VDE-requirements for electromedical devices DIN EN 60601, part 1 / DIN VDE 0750, part 1, too. It is suited for connection to supply lines installed according to the DIN VDE 0100-710 regulations for medically used rooms.

	This unit is not explosion-proof.
	Do not use in the presence of flammable anesthetic gas mixed with air or flammable anesthetic gas mixed with oxygen or nitrous oxide
	This dental treatment unit must not be modified or opened without permission of the manufacturer.
	We can only accept responsibility for the safety properties of our equipment, if installation, maintenance, repair and modifications are carried out by our workshop or by a person explicitly authorized by us. Further, the equipment has to be correctly handled according to our operating manual. Any unauthorized modification poses a great danger to the intended user as well as the patient.
	In case of failure, structural parts, that can affect the security of the device, are to be replaced by original parts only.
	In case of repair, the technician must issue a certificate as to the kind and extent of work performed and, if applicable, indicate any changes of nominal data and operating range. The certificate must contain the date of repair, the name of the executing company and a signature (according to MPDG, DIN VDE 0750 / DIN VDE 0751).
	Close to the equipment do not operate any devices producing electro-magnetic emissions.
	An ergonomic way of working must be chosen in order to prevent musculoskeletal disorders.
	Before using for the first time, check the attachment of the ground anchoring!

Safety-relevant inspections

The safety standards for medical products and all legal obligations are to be fully observed by the user.

Scope and schedules of safety-relevant inspections are as follows:

Every two years the following safety-relevant inspections according to DIN EN ISO 62353 have to be performed: see Form „Safety inspections – Dental treatment unit“.

We recommend to keep an equipment control log for documenting the results of the safety-relevant inspections.

The unit lifetime cycle is: 10 years.

	Multimedia For components like Intraoral camera, monitor, note the separate user manuals for these devices!
---	--

Disposal

According to law, electric equipment is to be disposed by authorized disposal firms only.

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Please always follow the following safety precautions to avoid injuries to persons or damages to the unit:

	WARNING!
	Always shut off the water supply, air supply and the main power before performing any cleaning or maintenance.
	Install the dental unit on level ground.
	Do not install the unit near radiators or heaters.
	Do not place or hang any heavy things on the unit.
	Do not connect unit with incompatible wires which may affect its performance, or even cause fire or electric shock.
	Do not place anything in the unit so as to avoid electric shock or fire. Do not spill any liquid into the inner part of the unit.
	Do not disassemble the unit so as to avoid electric shock. Contact technical service personnel whenever the unit needs repairing.
	WARNING: To avoid risk of electric shock, this equipment must only be connected to a supply mains with protective earth
	Do not open or remove the cover plate so as to avoid injuries caused by dangerous voltage or other dangers. An improper assembly may cause electric shock during the subsequent operation.
	Please switch off the power and seek help from technical service personnel if: (1) any break-down or wear-down of the plug or cable occurs. (2) unknown liquid spills into the inner unit or water flows into the inner circuits. (3) the unit malfunctions, even if it is operated according to the instructions.
	If possible, turn off the power supply during thunderstorms.
	Please never exceed this units weight limit (maximum load of the dental seat: 150kg).
	The maximum load of the table tray attached to the dentist table is 3.0 kg, which should never be exceeded.
	The manufacturer shall not be responsible for any damage or injury caused by mishandling, neglect or improper use.
Before leaving the surgery, shut off the water supply, air supply and the main power switch.	

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Translation of user manual

When imported into countries whose native language is not English or German, the operating manual are translated into local languages. In this translation, replace the words „original instruction for use“ by the following notice: „translation of the original instruction for use.“

 Ritter® Concept GmbH is not directly or indirectly liable for mistakes or failures in this documentation as well for the given benefits of the use with this operating instruction!

 The illustrations in this manual may be able to differ from the actual unit in your system.

 The duplicating of this documentation without written permission of Ritter® Concept GmbH is prohibited!

 Accompanying documents are written at a level consistent with education, training and other needs of individuals for whom they are intended.

Recycling

All packing material does not harm the environment and can be completely recycled.

- wooden base plate
- carton box
- polyethylene bag

Features and options – Stools

- Mobilo Flex 60
- Mobilo Flex 60 with footring (optional)
- Backrest with fixed position or dynamic movement
- Ergonomic and comfortable seating
- Symmetrical backrest
- Gas lift cylinder
- Chrome legs
- Five dual castors for stability
- Colour-matching upholstery available in multiple colours

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We recommend you to read this operating manual very carefully in order to have full knowledge of all the advantages this dental workplace is offering you.

The essential attributes of all models are:

- 5 outlets for turbine, air motor, electric micromotor, scaler or syringe

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Model "H" (hanging hoses)

- instrument carrier with hanging hoses smooth running swivel arm
- pneumatically limited
- integrated tray
- photoelectric sensor in Instrument holder

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Model "S" (swinging hoses)

- Instrument carrier with swinging lever
- smooth running swivelarm
- pneumatically limited
- electric switch in instrument holder
- maximum rotation angle of instrument tray: 180°

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Startup

The unit is activated via the power switch (1) at the patient chair base. After switching-on the control indicator of the power switch lights.

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It is recommended to actuate the hand pieces, the cup filler and the spittoon bowl rinsing before the first daily treatment is done in order to rinse out stale water.

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	<p>The power switch for switching the treatment unit on and off must not be blocked by any objects!</p> <p>It must always be possible to operate the switch to interrupt the power supply without any problems!</p>
--	---

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	<p>For safety reasons, switch off the main switch when leaving the surgery!</p> <p>Therefore press main switch (1). After switching-off, the control indicator of the power switch turns off. Use shut-off valves to shut off air and water inlet.</p>
--	--

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If a sudden power outage occurs switch off the main switch (1) and put all handpieces in their instrument holder. When electrical power comes back activate the unit via the power switch (1).

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Features and options – foot controller

for chair movement and instruments:

- large foot bow for easy repositioning
- manual control of chair movements

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Program control of chair movements:

- Spitting position/last treatment position
- two free programmable chair positions (A & B)
- Exit position

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Control of instruments:

- Speed/power control for the active instrument

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Buttons:

- Spray water at instrument
- Chip blower function

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	There is a risk of tripping due to the cable and the foot controller!
--	--

Control elements patient chair	
	Pedal / Foot lever
If all instruments (except the syringe) are in their holders.	
2	<p>Slide joystick to the <u>left</u> → Backrest DOWN</p> <p>Slide joystick to the <u>right</u> → Backrest UP</p> <p>Slide joystick <u>backwards</u> → Patient chair UP</p> <p>Slide joystick <u>forward</u> → Patient chair DOWN</p>
3	<p>press key <u>shortly</u>: Automatic movement to „Spitting position“, actua- ting again moves chair into last treatment position</p> <p>press key longer than <u>2 seconds</u> an acoustic signal sounds, Automatic movement to programmed „Position A“</p>
4	<p>press key <u>shortly</u>: Automatic movement to „Exit position“</p> <p>press key longer than <u>2 seconds</u> an acoustic signal sounds, automatic movement to programmed „Position B“</p>

Control elements for active instruments	
3	press key <u>shortly</u> : activates the spray water at instruments
4	with pressed key activates the chip blower-function
5	push lever to the right: starting the activated instrument and increase the speed/power
6	maximum speed/power of the instrument
7	Press the pedal. Start the active instrument and increase/decrease the speed/power

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Arm brake

Moving the dentist element:

- touch the dentist element at the handle (2)
- press the button (1) with your thumb and move dentist element in the desired position (heightwise)
- release button (1)

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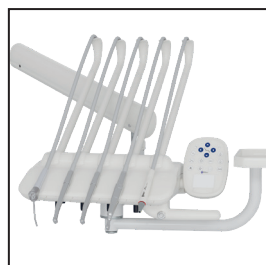
X-ray viewer

- Press the key „X-ray viewer“ to activate the X-ray viewer.
- It stops automatically after 3 minutes.
- Premature power off the X-ray viewer can be released by pressing key „X-ray viewer“ once more.
- Luminance $\approx 1600 \text{ cd/m}^2$

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Whip arm (dentist element S version)

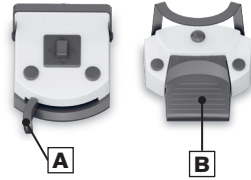
- take up an instrument and pull it forwards till a click sounds
→ then the instrument is ready for working
- after using, swing back the instrument on the instrument holder
(after a click sounds the instrument is deactivated)

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Startup:

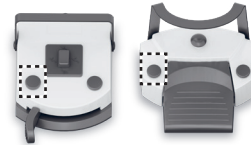
Take the hand piece from its holder.

The hand piece is activated via the lever (1) or pedal (2) of the foot controller.

Depending on the position of the lever (A) or the pressure on the pedal (B) – the power of the taken hand piece changes from a minimum to maximum power.

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
Spray water

To switch the spray water at the activated instrument ON/OFF: press left pushbutton. It is possible to set water ON/OFF for each instrument hose independently.

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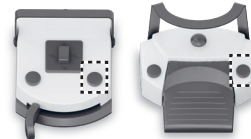Chip blower:

- switches an constant air flow at the activated instrument ON or OFF

As long as pressed the right pushbutton - the chip blower function is activated.

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Instrument light:

- take out the instrument from the holder
- press key shortly to switch the light at the handpiece ON/ OFF

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Setting the maximum rotation speed of micromotor

The maximum speed of the micromotor depends on the position of either the lever A or the pedal B on the footcontrol. Push lever A as far right as possible to achieve maximum speed. Analogous press pedal B as far down as possible.

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Reversal of rotation direction:

The switch (2) changes the direction of the rotation either into clockwise or anti-clockwise. It is located at the underside of the dentistelement

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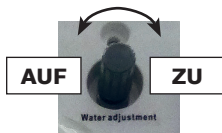
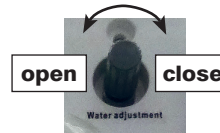
Spray water regulator for instruments

For each instrument (except the syringe) there is the possibility to regulate the spray water quantity by hand.

The regulators can be found at the bottom side of the dentist element (1).

The order of regulators depends on the order of the instruments on the table.

In order to regulate the water, take the according instrument from its holder and actuate it with spray water function. Now you can adjust the water quantity by turning the according regulator as illustrated in the picture below.

Spray regulator for all instruments

The regulator (1) is at the underside of the dentist element.

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Air pressure adjustment of handpiece:


There are air regulating valves used for adjusting the drive air for the instruments. They are placed at the underside of the dentist element.

1	Luftdruckregler für 1. Bohrinstrument	air pressure regulator for 1 st burr instrument
2	Luftdruckregler für 2. Bohrinstrument	air pressure regulator for 2 nd burr instrument
3	Luftdruckregler für 3. Bohrinstrument	air pressure regulator for 3 rd burr instrument

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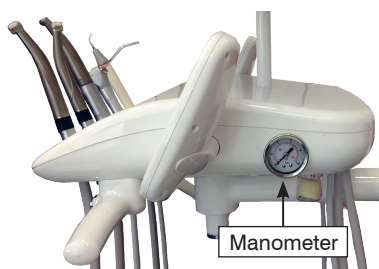
Adjustment drive air for handpiece:

Low speed pneumatic handpiece
No-load speed

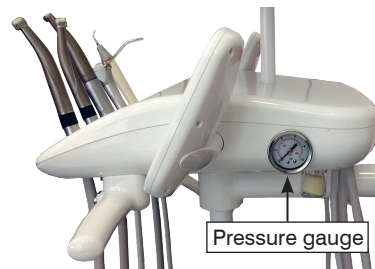- shall be no lower than 14000 rpm if the air pressure is 2.5 bar (250 kPa, impelling air)

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High speed turbine
No-load speed

- shall be no lower than 30000 rpm if the air pressure is 250 kPa (impelling air)
- a air pressure regulator is at the bottom of the dentist element
→ reduce the air pressure by rotating clockwise the regulator (2), (3) or (4)

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Enable/disable the prompt tone function

- press key  longer than 5 seconds to enable/disable the prompt tone function
→ a buzzer beep sounds
→ the prompt tone function of the key is enabled/disabled

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Scaler

Startup:

Take the hand-piece out of the holder.

The device is activated with the lever (A) or pedal (B) of the foot controller.

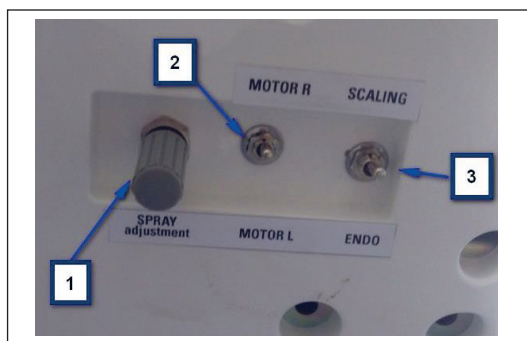
Intensity

The intensity of the scaler can be adjusted by pressing lever A or pedal B from minimum to maximum.

Scaler intensity range:

It is possible to choose the intensity range of the scaler using switch 3 at the underside of the dentist element. The intensity ranges from 100%, which is scaling mode, to around 50%, which is endo mode.

The range of intensity also depends on the used scaling tips. Please adjust the intensity accordingly.

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Adjust water volume of scaler

• by using the corresponding water regulator (1) at the underside of the dentist element
→ see page 28

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Note maximum power settings for using scaler tips!

→ see notes in instruction for use of scaler.

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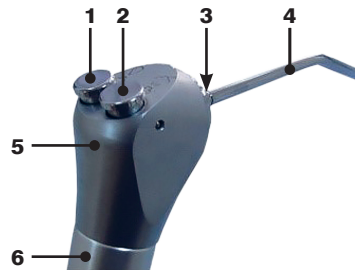
Functions:

Water, air or spray are switched by hand. Supply of the media lasts as long as the corresponding key is pressed. Either air or water can be obtained by actuating the corresponding key. Actuating both keys at the same time will supply spray.

Operation of 3F syringe:

- 1 Key for water delivery
- 2 Key for air delivery
- 1+2 Delivery of spray

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1	Key for water delivery
2	Key for air delivery
3	Spigot ring
4	Cannula
5	Three-way Syringe
6	Handle

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 Removable and autoclavable cannula (4).
Sterilization of cannula:
→ see page 53 (chapter Maintenance)

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Components:

1	Handle
2	Parabolic mirror
3	LED group
4	Proximity sensor

Operating light is fixed on flexarm mounted to the unit. It is equipped with two LEDs.

Power supply: 17-24V AC, 50/60Hz
 22V-35V DC

Power consumption: 9W (maximum)

Essential features:

- comfortable, shadow less light
- easy to move, keeps the adjusted position safely
- removable handle, sterilizable
- Electronic lighting regulation from maximum to minimum through a microprocessor
- no fan necessary

Startup:

The operating light can be turned on and off with the key „Operating lamp“ at dentist or assistant keyboard. Short press the button .

To switch on and off, move the hand near to the sensor (4) to a distance of about 3 cm.

Light intensity

Changing the light intensity via proximity sensor

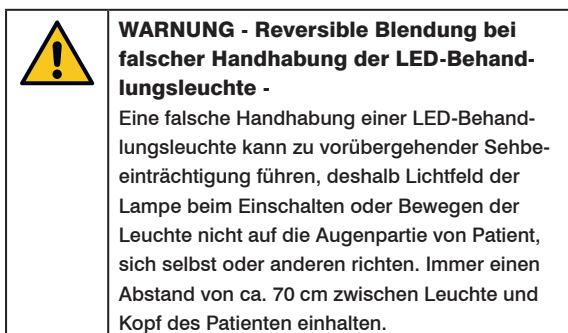
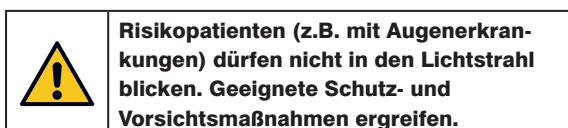
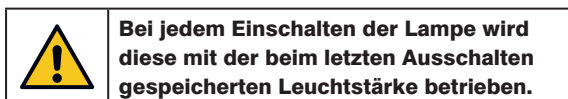
- To reduce the light intensity, remain with your hand near the sensor (4) until the desired level of intensity is reached.
- Luminosity can only be regulated from the maximum to the minimum level.

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**Change operating lamp handle**Remove handle

- hold the operating light with the left hand and hold handle (2) with the right hand.
- press the handle button (1) with the thumb and pull out the handle

Install the handle

- hold the operating light with the left hand hold handle (2) with the right hand
- press the handle button (1) with the thumb and insert the handle (2) to the spindle slot and loosen the handle

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The clearly geometry of reflectors, in combination with each LED source is, allows to realize a luminous flux which is homogeneous, clean and without shadows.

Essential features:

- comfortable, shadow less light
- easy to move, keeps the adjusted position safely
- removable handle, sterilizable
- dimmer
- no fan necessary
- adjustable color temperature

Control elements:

1	Handle
2	Regulator for brightness
3	Regulator for Colour temperature

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Light intensity

The required intensity can be adjusted individually by turning the dimmer (2).

Reduce light intensity

- Rotate the brightness regulator (2) anti-clockwise (rear view of the operating lamp) until it reaches the desired light intensity.

Increase the light intensity

- Rotate the brightness regulator (2) clockwise (rear view of the operating lamp) until it reaches the desired light intensity.

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Colour temperature

Adjustable range: 4.200 K - 6.000 K

Increase the colour temperature:
→ rotate the regulator (3) anti-clockwise

Reduce the colour temperature:
→ rotate the regulator (3) clockwise

Light spot

Size: 7 cm x 14 cm at a distance of 70 cm far from the source and with well defined outlines, in order to avoid risk of patient dazzling.

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 **Each time the lamp is turned on, the light intensity will be at the level memorised when it was turned off the time before.**

 **Patients at risk (i.e. with eye conditions) are not allowed to look into the light beam.**

 **WARNING - reversible dazzling with wrong use of LED operating lamp -**
A wrong use of the LED operating lamp can lead to temporary visible disturbance, therefore, when turning on or moving of lamp, never direct light field to eyes of patient, yourself and others. Always ensure a distance of approx. 70 cm between lamp and head of patients.

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Operating light is fixed on flexarm mounted to the unit.

Essential features:

- comfortable, shadow less light
- easy to move, keeps the adjusted position safely
- removable handle, sterilizable
- second light modus with yellow light (working with photosensitive resin)
- color temperature: 5000 K

Control elements:

1	Handle
2	Sensor
3	Sliding-switch for colour temperature

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To switch on and off, move the hand near to the sensor (2) to a distance about 5 to 12 cm.

Light intensity

Increase the light intensity

- Place the hand near the sensor (2) until desired intensity is reached. (Luminance increase from minimum to maximum occurs in 4 steps. After reaching the maximum luminous intensity, the lamp switches back to minimum light intensity.)

Blue cut-off function:

The dental operating light has blue cut-off function.

While using photosensitive resin to fill the teeth, adjust the color temperature of operating light into a yellow light (without blue light emission). So it is possible to prevent fast curing of photosensitive resin.

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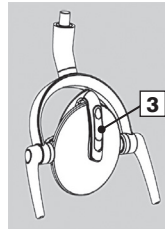
Standard light (at a regular oral treatment):

- move sliding switch (3) **up** and lock it properly

Yellow light (treatment with photosensitive resin):

- move sliding switch (3) **down** and lock it properly

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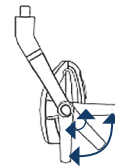
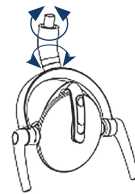
Adjust operating lamp

Three active joints are available on the dental operating light which can adjust the desired angle.

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
Change operating lamp handle

Remove handle

- hold the operating light with the left hand
→ hold handle (2) with the right hand.
- press the handle button (1) with the thumb and pull out the handle

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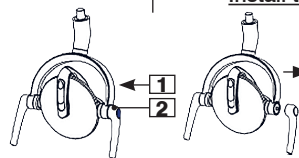	Each time the lamp is turned on, the light intensity will be at the level memorised when it was turned off the time before.
--	--

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	Patients at risk (i.e. with eye conditions) are not allowed to look into the light beam.
--	---

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	WARNING - reversible dazzling with wrong use of LED operating lamp - A wrong use of the LED operating lamp can lead to temporary visible disturbance, therefore, when turning on or moving of lamp, never direct light field to eyes of patient, yourself and others. Always ensure a distance of approx. 70 cm between lamp and head of patients.
--	--

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1	Big suction
2	Curing light
3	Assistant keyboard
4	Syringe
5	Small suction

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Features and options – Assistant element

- moveable assistant element with assistand keyboard
- 3 functional syringe
- big and small suction hose
- Polylight LED (optional)

Features and options - water unit

- swivelable ceramic cuspidor
- ABS Aqua Bottle System
- Heating for cup filler
- Connection for external suction (optional)
- Place selection valve (optional)
- Venturi suction system

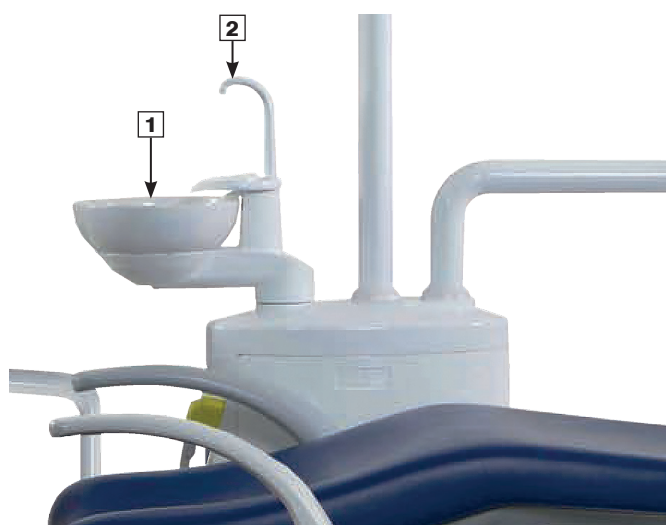
Abb.: Speischale (1), Spülglasfüller (2)
Fig.: Spittoon bowl (1), Cup filler (2)

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Spittoon bowl rinsing

The spittoon bowl rinsing starts when pressing the key „**spittoon bowl rinsing**“ and stops automatically after a programmed time.

Premature stop during bowl rinsing is possible by pressing the key once more.

Programming of spittoon bowl rinsing time:

(This is only possible at the dentist keyboard)

- press key  longer than 5 seconds
→ the indicator LED flashes
- press key  shortly
- when the desired rinse time is reached, press key  again to finish the spittoon bowl rinsing
→ indicator light turns off and the programming mode is terminated

Cup filler

The cup filling starts when pressing the key „**Cup filler**“ and stops automatically after a programmed time.

Premature stop of cup filling can be achieved by pressing key „Cup filler“ once more.

The recommended filling level is two thirds of the glass or plastic cup. Overflowing water will run off through the cup support plate to the drain.

Programming of cup filling time

(This is only possible at the dentist keyboard)

- press key  longer than 5 seconds
→ the indicator LED flashes
- press key  shortly
- when the desired filling level is reached, press key  again
→ indicator light turns off and the programming mode is terminated

Cup fill heating

- press key  shortly
→ the indicator LED flashes
When the water reaches the preset temperature of 34°C the indicator lights continuously.

1	Spraysauger
2	Speichelsauger

1	Big suction
2	Small suction

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Big suction (1)

- take out the suction handpiece
→ suction starts automatically
- put the suction handpiece back into the instrument holder
→ suction will stop automatically after 3 seconds

Small suction (4) (small suction)

- take out the suction handpiece
→ suction starts automatically
- put the suction handpiece back into the instrument holder
→ suction will stop

 **Suction hose and suction handpieces are expendable materials.**

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Important safety regulations

 **The bottle is pressurized! (ca. 2.0 bar)**

 **Don't manipulate the bottle with sharp objects!**

 **Use only original bottles from Ritter® Concept GmbH.**

 **Before opening the bottle it is necessary to check that the switch (Bottle ON/OFF) is set to **Off** (see page 44, Fig. 2).**

 **After filling the bottle with water always screw the bottle in the correct way into the bottle holder!**

 **The bottle must be screwed only hand-tight to the bottle support.**

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After disinfection, clean the thread of the bottle and bottle holder regularly.

If the bottle has a mechanical damage, the bottle must be replaced by a new bottle.

For safety reasons, the use of the bottle is limited to a period of **6 months**. After that the old pressure bottle must be replaced by a new one.

In non-compliance with these safety regulations the company Firma Ritter® Concept GmbH shall not be liable for any resulting damages!

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Water-bottle-system

The water-bottle-system consists of:

- water bottle (for distilled water)
- switch (at the bottom of the water unit)

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Change water bottle

Set the switch for bottle compressed air (1) to OFF position („Bottle OFF“) (Switch is located at the bottom of water unit.)

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Remove the empty bottle out of the bottle holder (screw bottle anti-clockwise)
Fill the bottle with distilled water
Screw the filled bottle into the support (clockwise, do not screw too firmly!)
Set the switch for bottle compressed air (1) to ON position („Bottle ON“, Bottle is now under pressure)

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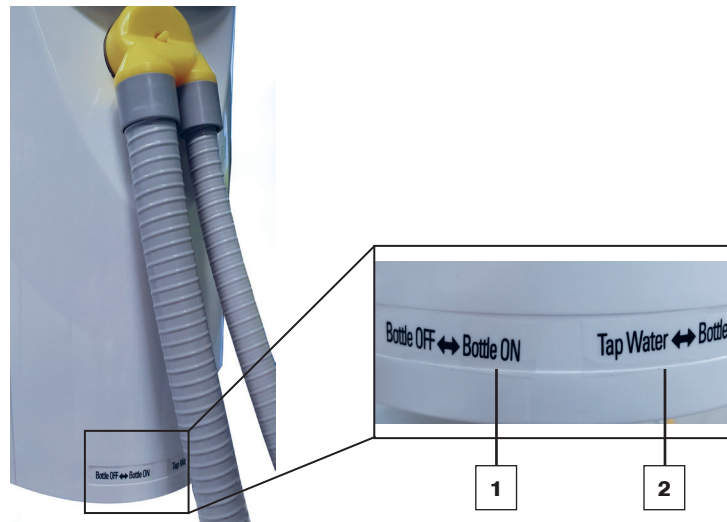
Bottle can only be unscrewed when it is depressurized.

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1	Air switch - Bottle ON/ OFF compressed air <ul style="list-style-type: none"> • „Bottle OFF“ → Bottle in pressure-free mode • „Bottle ON“ → Bottle is now under pressure
2	Toggle - water source <ul style="list-style-type: none"> • „Tap water“ → water from water pipeline • „Bottle“ → water from bottle

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 **After switching to „Bottle“, only the dentist table will be supplied with water from the bottle.**

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	<p>We strongly recommend to use distilled water. If there are impurities in tap water, it is necessary to use the distilled water from bottle.</p>
--	---

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Patient chair

Features and options – patient chair

- additional counter-movement of seat while moving the backrest (Trendelenburg-movement) increases the comfort and avoids the "undressing effect"
- quickset programming for five chair positions
- double articulated headrest, mechanically
- left and right armrest for patient
- seamless upholstery
- Operation of patient chair can be done manually by hand via dentist or assistant keyboard or by foot via multifunctional foot controller
- safety switches

Ritter Concept GmbH
Freiburger Straße 45
88400 Biberach, Germany
Fon: +49 7351 52 925-10
Fax: +49 7351 52 925-11

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When moving the back rest, the arms of the patient must not be placed behind his back!
Otherwise there is a risk of squashing!

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Safety switch

If the patient chair or backrest moves downwards and hits an obstacle, the movement stops automatically and it moves upwards a little.

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
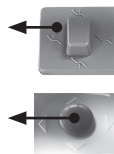
Slide joystick to the left
→ backrest DOWN

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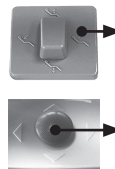
Slide joystick to the right
→ backrest UP

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Slide joystick upwards
→ patientchair UP

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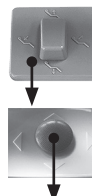Slide joystick downwards
→ patientchair DOWN

Control elements

The patient chair can be controlled from the dentist keyboard or with the foot controller.

Important:

 If any instrument is taken from its holder (except syringe) and is activated using the foot control, the control of the patient chair is no longer possible.

Manual control via dentist or assistant keyboard:

The patient chair is controlled manually by keys

- "Backrest UP"

- „Backrest DOWN“

- "Patient chair UP"

- „Patient chair DOWN“

and can be brought into any desired position. The arrows on the keys indicate the direction of the chair movement.

Manual control via foot controller:

The buttons of the foot controller have following functions:

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Wichtig:

 **Any automatic patient chair movement will be stopped by pressing any key.**

Programming the chair positions:

The programming of the automatic chair positions can be done by using the dentist keyboard.

Please proceed as follows:

Treatment position A

- move the patient chair manually into the desired position
- actuate and keep actuated key „Treatment position A“  on the dentist keyboard longer than 5 seconds till you can hear a single beep
(This beep indicates, that the programming of the position was successful.)

Treatment position B

- move the patient chair manually into the desired position
- actuate and keep actuated key „Treatment position B“  on the dentist keyboard longer than 5 seconds till you can hear a single beep
(This beep indicates, that the programming of the position was successful.)

Exit position

- move the patient chair manually into the desired position
- actuate and keep actuated key „Exit position“  on the dentist or assistant keyboard longer than 5 seconds till you can hear a single beep
(This beep indicates, that the programming of the position was successful.)

Emergency position

- move the patient chair manually into the desired position
- actuate and keep actuated key „Emergency position“  on the dentist or assistant keyboard longer than 5 seconds till you can hear a single beep
(This beep indicates, that the programming of the position was successful.)

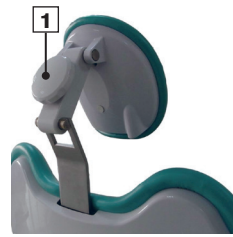
Spitting position

- move the patient chair manually into the desired position
- actuate and keep actuated key „Spitting position“  on the dentist or assistant keyboard longer than 5 seconds till you can hear a single beep
(This beep indicates, that the programming of the position was successful.)

Setting of linkage between spitting position and spittoon bowl rinsing:

- actuate and keep actuated key „SET“  on the dentist or assistant keyboard longer than 5 seconds till the indicator light beside the the key  is on
- actuate key „spittoon bowl rinsing“ 
→ a signal sounds and the indicator light goes out
→ spittoon bowl rinsing at spitting position is activated

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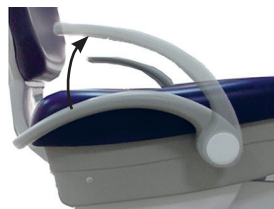
Adjust the headrest:

The double joint can be adjusted by the knob (1):

- rotate the clamping knob (1) anti-clockwise to loose joints
- move the headrest manually into the desired position
- rotate the clamping knob (1) clockwise to tighten it

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
Move right armrest up

- moving the armrest upwards, until it clicks

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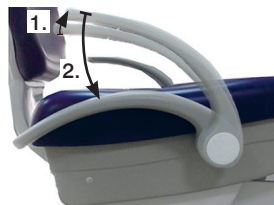Move armrest down

- move the armrest upward to the highest position
- then the handle is movable to the lowest position

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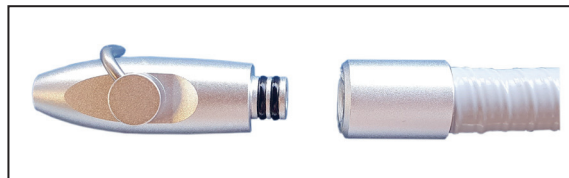
Conformity with the hygienic standards as well as good performance of the equipment are warranted by observance with the instructions to the recommendations for disinfection, cleaning and maintenance.

Burr instruments:

- Every day before the first treatment, let instruments run for approx. 120 seconds to purge all spray tubes.
- Clean and maintain turbine, handpieces and contra-angle attachments.


	IMPORTANT: Please read the manufacturer's operating manual for the burr instruments very carefully.
---	--

	IMPORTANT: Instrument hoses may not be cleaned in the thermodesinfektor. Temperature load for the tubes is max. 40°C
---	---

Suction handpieces:

Detach handpiece from hose. Clean accessible hose ends and handpieces. Lubricate O-rings with vaseline and reassemble handpiece to the hose.

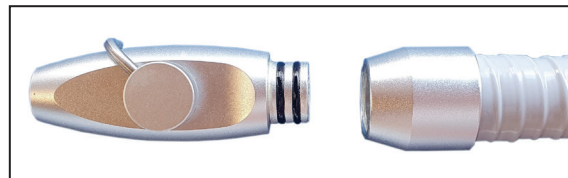Unsuited and not permitted are cleaning agents based on quaternary ammonium compositions, cleaning agents containing phenol and aldehyde as well as foam generating agents.

	IMPORTANT: For dosage directions, refer to the recommendations given for the individual cleaning and disinfection agents.
---	--

Small suction

Small suction should be equipped with disposable suction nozzle.

- Throw away the suction nozzle after treating each patient and replace with a new one before the treatment of next patient.

Big suction

- sterilize Big suction nozzle by high-temperature steam sterilization (134°C, 15 min)

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Cleaning the filter (suction)

Saliva suction

Cleaning suction hose and suction filter

- take out the suction handle from instrument holder
- pull out the suction hose from the suction filter
- take out the filter screen from the filter and clean it with clean water
- install suction handpiece
- install filter screen into the filter

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Cleaning suction system:

- after each patient: empty a glass of clean water with each small and big suction hose
- one/two times a day: suck clean water or sterilized water for about 30 seconds

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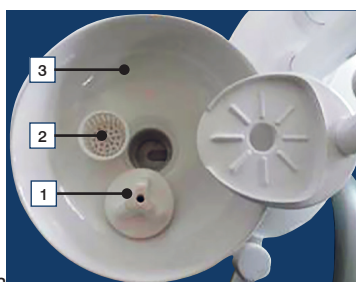
Water filter

The unit has water filters to ensure its normal use.

After serving a certain time, dirt may block the filter core and affect the water flow, therefore, it is necessary to clean and replace the filter core.

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
Spittoon bowl

- take out the mushroom (1) and filter screen (2) from the spittoon bowl
- clean the mushroom (1), filter screen (2), spittoon bowl (3) and spittoon bowl holder with water and disinfect it with alcohol-free solution
- install the spittoon bowl (3)
- install filter screen (2) and mushroom (1) into the spittoon bowl

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

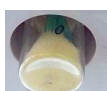
Turbine exhaust air oil container

- Screw off the container anti-clockwise, empty the turbine exhaust air oil container (1) and replace cotton wool.
- Screw on the container clockwise
The container is located at the bottom side of the dentist unit

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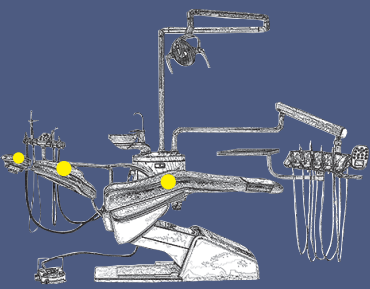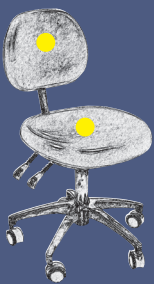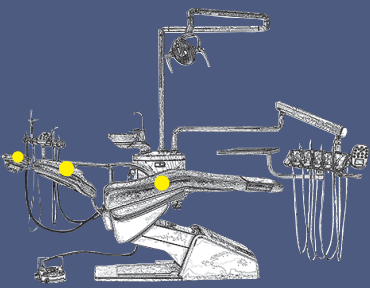Cleaning and care instructions

What	How	Wherewith	Notes
	<p>● Clean/disinfect upholstery Wipe upholstery, wiping disinfection</p> <ul style="list-style-type: none"> • Headrest upholstery • Backrest upholstery • Seat upholstery 	<p><u>Cleaning:</u></p> <ul style="list-style-type: none"> • Soft sponge, lukewarm water, soap. • Clean heavily soiled areas with a soft brush. <p>After being cleaned, the upholstery should be wiped dry with a cloth.</p> <p><u>Disinfection:</u> We recommend applying alcohol-free disinfectant spray foam for cleaning and disinfecting imitation leather upholstery to your upholstery.</p>	<p>Remove soap residue with a damp sponge and dry it with softy, lint-free cloth.</p> <p>A cloth is not suitable for cleaning! Never use cleaning agents or household cleaners.</p> <p><u>Discolouration</u> In most cases, discolouration stains that are detected immediately can be removed with the right cleaning tools and cleaner for imitation leather.</p> <p>Discolouration stains that are not detected immediately cannot be removed.</p> <p>Protect the upholsteries from direct sunlight.</p>
	<p>● Clean/disinfect upholstery Wipe upholstery, wiping disinfection</p> <ul style="list-style-type: none"> • Backrest upholstery • Seat upholstery 		

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**Keep liquids away from electrical connections and components!**

- Most electrical control switches ARE NOT hermetically sealed.
- When a switch gets wet, it can self-activate, or perhaps short out tad damage other system components!
- Keep switches dry.
- Avoid costly equipment damage or failure.
- Please do not spray disinfectants directly at or into electrical switches and controls.
- Simply spray these solutions onto a cloth (not saturate the cloth) and wipe down the control surfaces.

Any not recommended maintenance/change by the manufacture may cause risk of injury or electrical shock!

- Don't let more than one patient sit in the chair at a time
- Don't stand anywhere on the dental chair
- Don't operate the dental chair around any flammable liquids.

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Handpiece

Handpiece is a delicate device, so it shall not be pounded or dropped.

It shall be cleaned and lubricated with special detergent and lubricant every day. As for the specific method, refer to the instruction manual of the handpiece.

Oil spray lubrication shall be conducted before high temperature sterilization.

Please pay attention to the operating manual of the hand piece!

Operating lamp „Standard light LED“

Handle

- Changing
→ see description on page 33.
- Cleaning/Disinfection:
→ neutral cleaning solution, alcohol-free disinfectants

Lamp housing

- Cleaning/Disinfection:
→ neutral cleaning solution, alcohol-free disinfectants

Operating lamp „SunLite“

- → see instruction for use of operating light „SunLite/Polaris“

Operating lamp „CoolLite“

- → see instruction for use of operating light „CoolLite“

Cannula of Syringe

- sterilization in steam sterilizer: at 134 °C for 3 min

(to remove the nozzle, press spigot ring (see page 27) and remove the nozzle for sterilization)

Suction handpiece

- Cleaning/Disinfection:
→ neutral cleaning solution, alcohol-free disinfectants
- sterilization in steam sterilizer: at 134 °C for 3 min

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
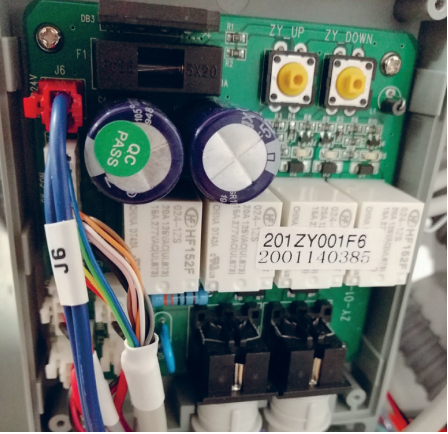
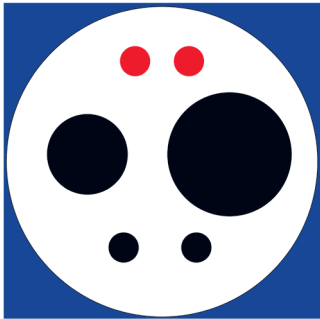
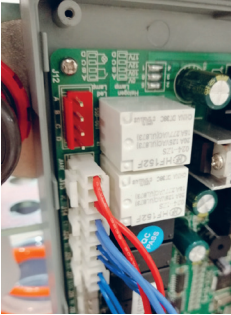
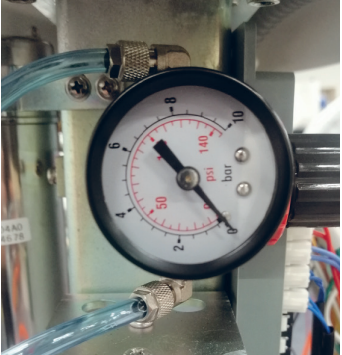
Overall failure



Failure	Troubleshooting
Failure 1: After power-on, the overall system makes no response.	<ol style="list-style-type: none"> 1. Check whether AC 220 power supply output is in good condition. 2. Check whether the 10A fuse at power switch is broken.
Failure 2: After power-on, no water or air is supplied to the system.	<p>Check that water source switch is in the correct position and check that pressure regulators are correctly set:</p> <ul style="list-style-type: none"> • If water source switch is set to tap water, put air switch - cylinder pressure in OFF position („Bottle OFF“);when water source switch is in “Bottle” water position, the pressurizing/pressure relief switch should be in pressurizing position. • Check inlet pressure on main air pressure regulator: it needs to be at least 5.5 bar. • In case unit is equipped with main air-water inlet group: check if 24 V_{DC} are present on both air and water main solenoids, also check if they are open.



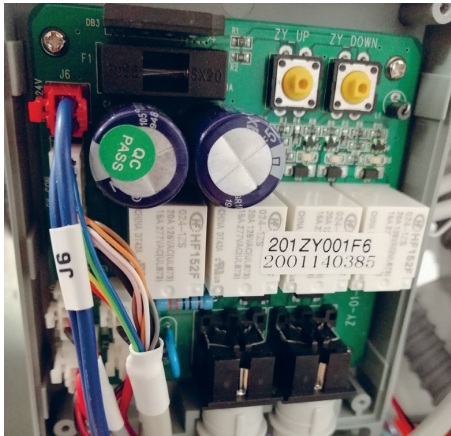
Common Failures

Failure	Troubleshooting
Failure 1: Operating chair fails to ascend or descend	<p>Check whether there is a voltage 24V AC input at port J6 of chair control board.</p> <ol style="list-style-type: none"> 1. If there is AC 24V voltage input at J6 port, check whether the two yellow keys can operate to achieve ascent/descent of chair; if not, the control board is likely to have been damaged, replace the control board to solve the problem. 2. If there is not AC 24V voltage input at port J6, check whether the chair power supply plug on flat cable is in good connection and connected properly. If not, tighten the screw. 3. Check fuse on chair board. 
Failure 2: Operating chair ascend or descend automatically	<p>Inspections: In case that operating chair ascend or descend automatically, perform troubleshooting from the following aspects:</p> <ol style="list-style-type: none"> 1. Assistant Section Touch Control Board: The assistant section control board may be corroded due to damp absorption or entry of water, and thereby cause malfunction of the function keys. Inspection method: Disconnect the communication cable terminal on the control board, operate the machine by using pedal and treatment platform to check whether the failure continues; → if the failure is removed, it indicates that the control board is damaged, and replace it; → if the failure continues, re-connect the communication cable terminal, and go to the next step. 2. Treatment platform touch control board: The treatment platform control board may be corroded due to damp absorption or entry of water, and thereby cause malfunction of the function keys. Inspection method: Disconnect the communication cable terminal on the control board, operate the machine by using pedal and treatment platform to check whether the failure continues; → if the failure is removed, it indicates that the control board is damaged, and replace it to remove the problem.

Failure	Troubleshooting
<p>Failure 3: Water heater fails.</p>	<p>If the water heater is without function, please contact a qualified technician. It is not allowed to replace the water heater without a qualified technician.</p>
<p>Failure 4: Handheld sprayer operates normally but no water or air is sprayed out.</p>	<p>Inspections:</p> <ol style="list-style-type: none"> 1. Check whether the active instrument operates currently in dry grinding mode where no water or air is sprayed by operating the change-over switch on it. For more details, see the operating instructions of the handpiece. 2. Check the water/air hose line is in good connection in instrument panel. If the hose line is in good condition, then check whether there is 24 V_{DC} output at water/air circuit solenoid valves. If yes, the solenoid valve may be broken, and needs to be replaced; if not, measure voltage output at reserved port J33, if there is a voltage output, replace treatment platform control board, if not, inspect the power supply in side case 3. Foot-operated micro switch: Check whether the foot-operated micro switch resets normally; if not, replace the micro switch. <p>Replace side case control board</p>
<p>Failure 5: Programmable chair storage function fails</p>	<p>Inspections: Generally, failure of programmable chair storage function is a result of damage to the angle sensor. Replace the corresponding angle sensor.</p>
<p>Failure 6: There is water or oily dirt in air hose</p> <p>Failure 7: LED on turbine is not working</p>	<p>Solution: Check air supply source and air filter; clean or replace relevant components.</p> <p>Remove instrument hose from its holder and press footcontrol to max. Check if the button for LED lighting on instrument is active on dentist element. If the button is active, measure LED voltage at the end of the hose. It needs to be 3.3 V_{DC} at the two red marked pins in the picture below. If there is no voltage present, please check connection on the table board. Also check if the hose is maybe broken.</p> 
<p>Failure 8: Micro motor is not running</p>	<p>On BMC 40 board green LED should be permanently ON. When foot control is pressed this LED should start blinking. Measure on pin 1 and 2(from left side on picture), voltage should be in the range of 0-5V in the moment of pressing foot control, depending on how much you press the pedal down. If there is no voltage check if cable from pin 1 and 2 is connect to J36 on table board If his is ok then change foot control and check again; next change water unit board and check again, next change table board and check again .</p> <p>On pin 9 and 10 it needs to be 32 V_{DC} continuously. If not then check 24V_{AC} on pin 1 and 2 of CONV24V. If not check 24V_{AC} on J33 table board. If not check all fuses in water unit board.</p>

Failure	Troubleshooting
<p>Failure 8 (cont.)</p>	<div data-bbox="684 308 1094 889" data-label="Image"> </div> <p data-bbox="671 912 809 943">BMC40 board</p> <div data-bbox="690 950 1099 1347" data-label="Image"> </div> <p data-bbox="671 1359 821 1390">CONV24 board</p> <div data-bbox="690 1397 1099 1794" data-label="Image"> </div> <p data-bbox="671 1806 941 1836">Fuses to check in water unit</p>
<p>Failure 9: LED on micro motor is not running</p>	<p>LED needs to be ON in moment of running micromotor. If motor is running and LED not working, then the LED needs to be changed. When replacing LED on motor take care of polarity position.</p>

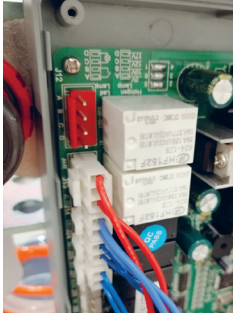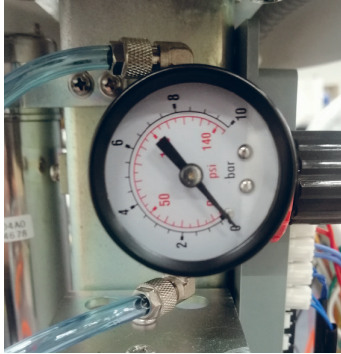Failure	Troubleshooting
<p>Failure 10: Scaler not working or not enough working power</p>	<p>Under dentist table there is switch for changing between scaling and endo mode. For maximum power it needs to be in scaling mode. Measure the voltage on J55 of table board when you are pressing the foot control to maximum while using the scaler; it should measure 0-5 V_{DC}. If there are 5 V_{DC} present, please check J51 for start switch of the scaler. If this is also ok change the handpiece first and then scaler board.</p>
<p>Failure 11: Operating lamp is not working</p>	<p>After starting up the unit for the first time, press the symbol for the operating lamp on the dentist element. You should hear a 'click' of the relay inside. If the lamp still isn't working please measure connector J12 on the water unit board. It should be 12-17V. If there is no voltage present, check the fuses on the water unit board.</p> 
<p>Failure 12: Small amount of water on cup filling and spitton rinsing after opening small regulator each line</p>	<p>Open and clean water filter or replace if need. Check hoses from solenoid valve to cup filler and spittoon rinser</p>
<p>Failure 13: Water pressure on instrument/syringe is too big or too low</p>	<p>Water source switch needs to be in BOTTLE position and under pressure. On this pressure regulator it is necessary to set 2 bars for adequate water pressure</p> 
<p>Failure 14: Dentist table is falling down with more items on it</p>	<p>Open this cover on instrument arm and increase force in instrument arm by fixing this screw clockwise.</p> 

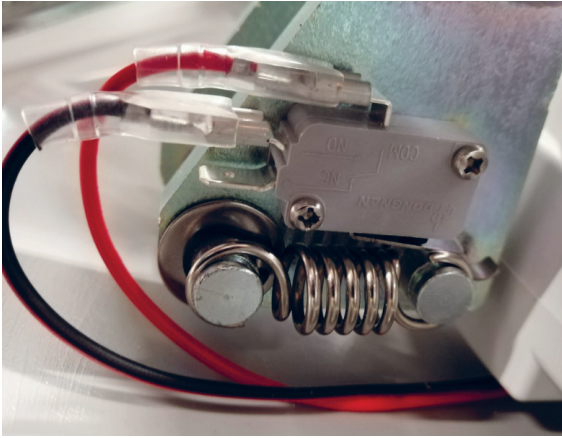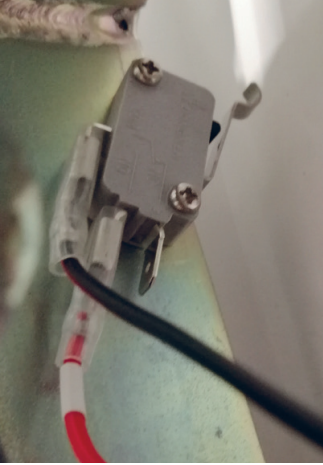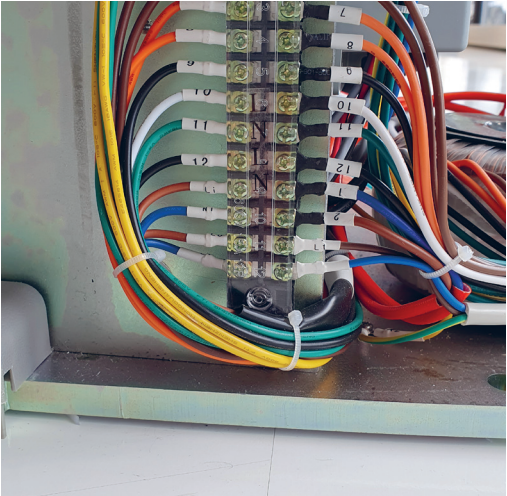Failure	Troubleshooting
<p>Failure 15: Safety switch of chair is not working</p>	<p>Backrest safety switch need to be connected like this and in normal condition is NO (normally open) and when it is pressed in closed position (contact).</p>  <p>Patient chair safety switch needs to be connected like this and in normal condition it is NO (normally open) and when it is pressed in closed position (contact)</p> 
<p>Failure 16: Treatment unit doesn't start up when switching on using main power switch</p>	<p>Check the fuses and your ground fault circuit interrupter. If both of them are working properly the main power cord may be broken. Only authorized technicians and personell are allowed to change the main power cord on the dental treatment unit!</p> <p>Remove the front cover of the patient chair and carefully unplug the power cord from the outlet.</p> 

Failure	Troubleshooting
Failure 16 cont.:	Unscrew the power cord from the terminal strip on pin 11 and 12. Reconnect the new power cord and test by switching the unit on.

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- Check the electric supply every month.
- Water pressure and air pressure must be in accordance with the unit requirement.
- Keep the hoses smooth and unblocked.
- Check the chair's movement and control panel.

When required

- Water inlet filter:
cleaned or replaced the water filter element, if:
 - the filter is used after a year
 - the filter collector is contaminated
 - the water coming out becomes turbid

Replace filter element

- open the water unit
- remove the water filter
- remove the filter element anti-clockwise
- take off the sealing and the collector successively
- clean or replace the filter element
- put in the collector and the sealing
- insert the filter element anti-clockwise
- check the sealing for leakage
- close the water unit

Automatic self test:

- switching-off the dental unit at main switch (control indicator of the power switch turns off)
- press button  and keep pressed while switching-on the dental unit at main switch
- release the button  after an indication of beep

- the patient chair will move up and down automatically
- operating lamp will turn on
- the heater will start

Press any key to stop the automatic self test.

Reset the programmed chair positions to factory settings:

- press key  for 5 seconds (An indication beep sounds, the chair will first move up to the highest position then down to the lowest position. Now all programmed chair positions are reset to factory settings.)

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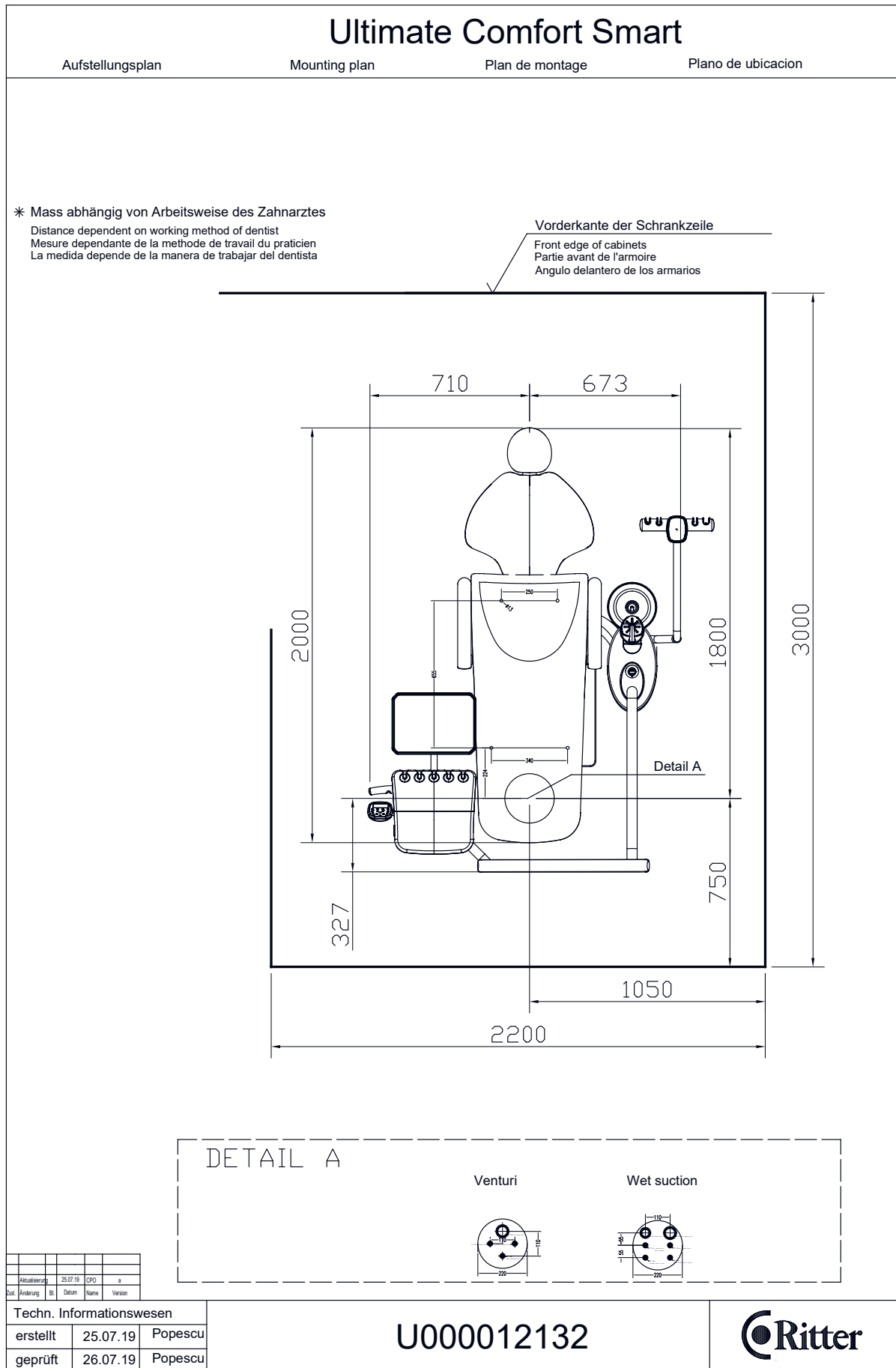
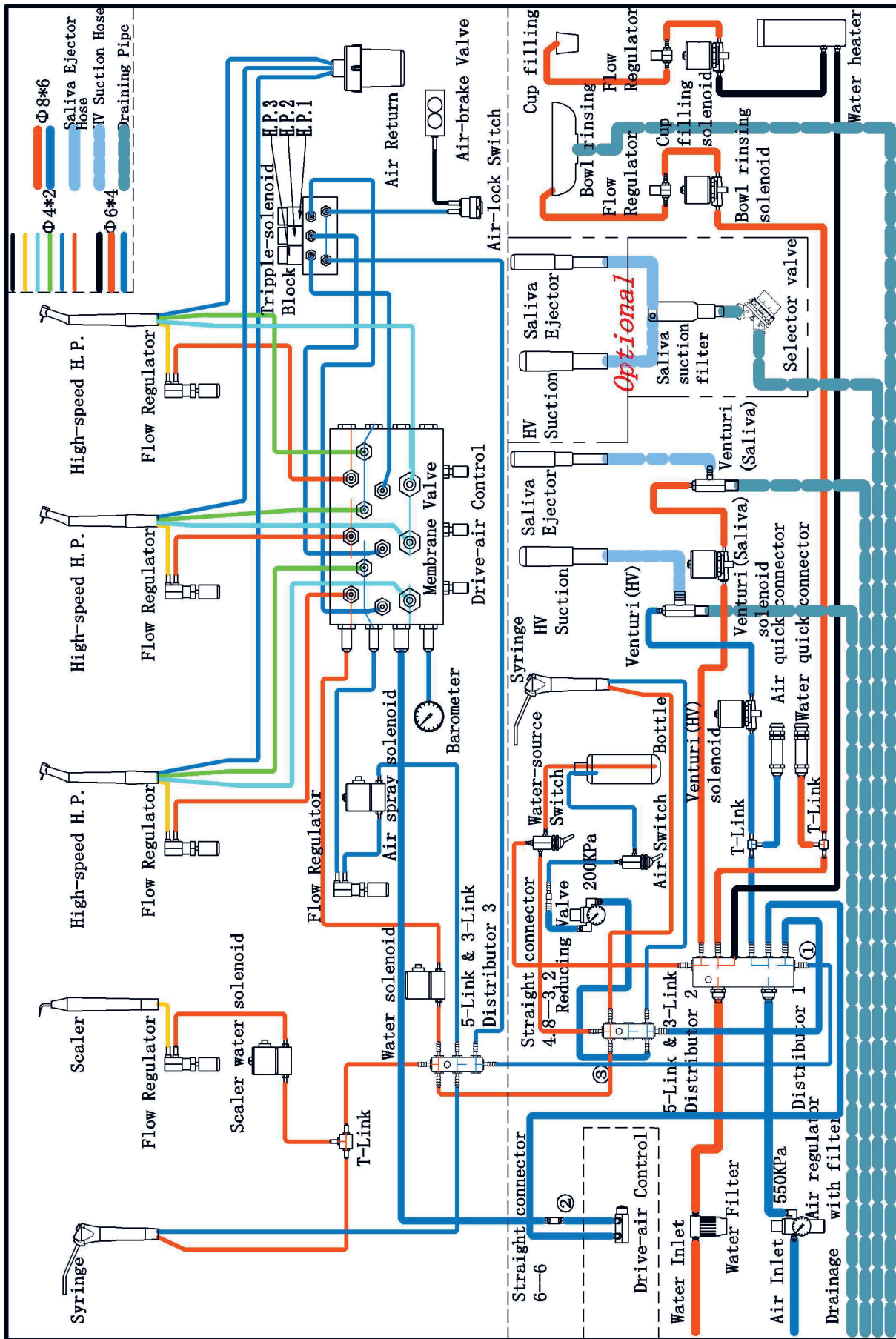
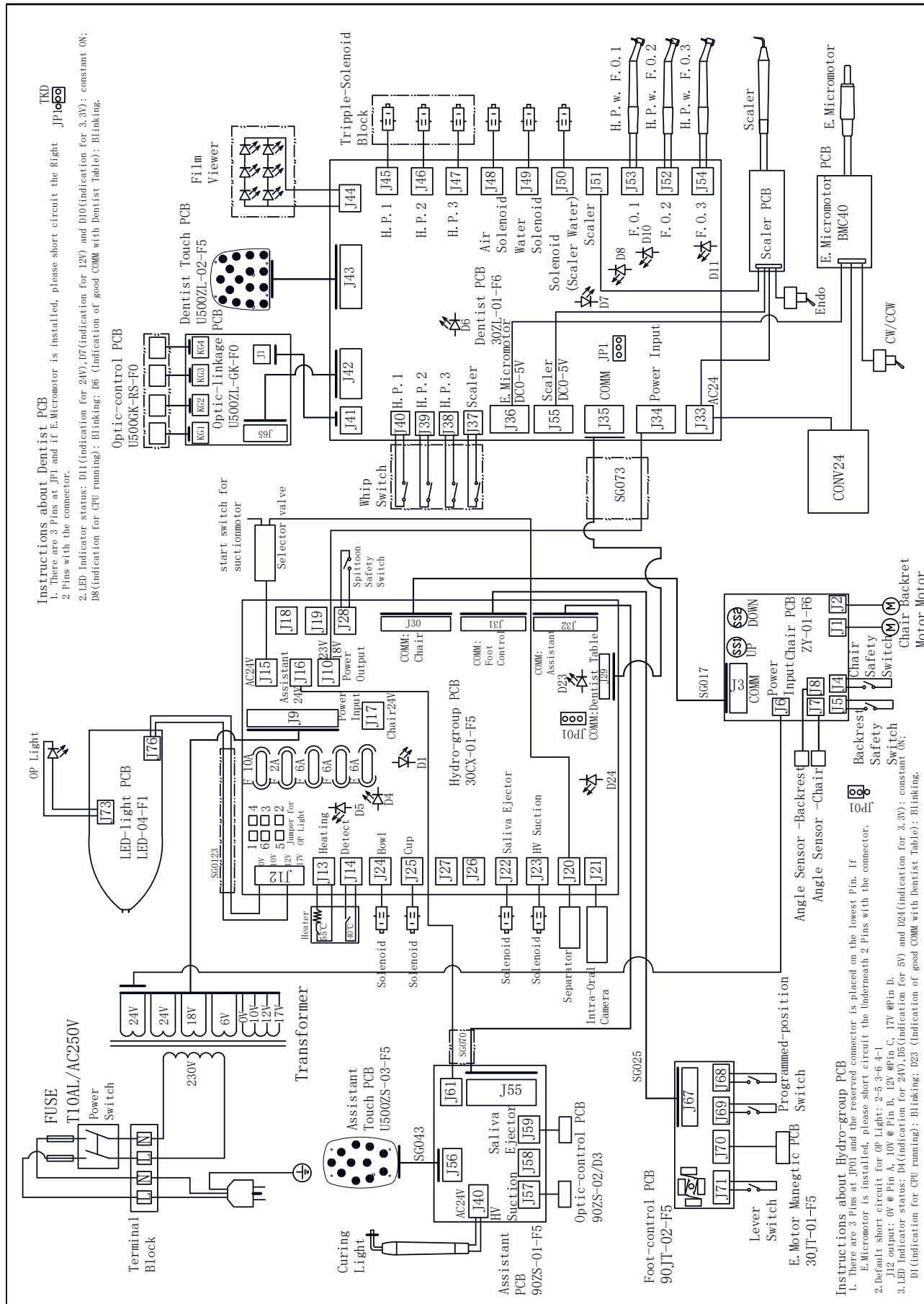
Inspection protocol for safety inspection according DIN EN 62353			
Client (dental practice)		Contractor	
Device		Used measurement device	
Device type:	Dentale Behandlungseinheit	Device type:	Electrical Safety Tester
Manufacturer:	Ritter Concept GmbH	Manufacturer:	
Type:		Type:	
Serial No:			
Netzspannung:V	Insulation class:	1
Safety inspection according:	EN 62353/VDE 0751	Type of applied parts:	Typ BF
Visual inspection			
<input type="checkbox"/> Protective earth conductor not damaged (for insulation class 1 only)			
<input type="checkbox"/> Cover and mechanical parts not damaged			
<input type="checkbox"/> Insulating parts not damaged			
<input type="checkbox"/> Device connecting cables including plugs without defects			
<input type="checkbox"/> Labels exist or completed, nameplate correctly			
<input type="checkbox"/> Others			
Measurement			
		measured value	limit value
			O.K./not O.K.
Protective earth resistance			<0,3Ω
Insulating resistance			>2 MΩ
Test voltage			~500 V
Device leakage current (Equivalent device leakage current)			<10 mA
Encl. leakage current (Housing leakage current)			<5 mA
Leakage current of application part (Equivalent patient leakage current)			<5 mA
Test voltage leakage current (nominal Voltage ±10%)			min.: -10% U _{Netz} max. + 10% U _{Netz}
Functional testing			
<input type="checkbox"/> Function and safety testing without defects			
<input type="checkbox"/> The unit has safety deficiencies:			
<input type="checkbox"/> Short-circuit L-N			
<input type="checkbox"/> Body circuit L/N-SL			
Test cycle: months		
Next test date:		
Tester:			
.....			
Place, Inspection date	Signature		

Ultimate Comfort Smart

Guidance and manufacturers declaration - electromagnetic emission		
The Ultimate Comfort is intended for use in the electromagnetic environment specified below. The customer or the user of the model Ultimate Comfort should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions according CISPR 11	Group 1	The Ultimate Comfort uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment. The Ultimate Comfort system is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
RF emissions according CISPR 11	Class B	
Harmonic emissions according IEC 61000-3-2	Class A	
Voltage fluctuations/flicker emissions according IEC 61000-3-3	Compliant	

Guidance and manufacturers declaration - electromagnetic immunity			
The Ultimate Comfort is intended for use in the electromagnetic environment specified below. The customer or the user of the Ultimate Comfort should assure that it is used in such an environment.			
Immunity tests	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) according IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst according IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	±2 kV for power supply lines ±1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge according IEC 61000-4-5	±1 kV differential mode ±2 kV common mode	±1 kV differential mode ±2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Interruptions and voltage variations on power supply input lines according to IEC 61000-4-11	<5% U_T (>95% dip in U_T for 0.5 cycle) 40% U_T (60% dip in U_T for 5 cycles) 70% U_T (30% dip in U_T for 25 cycles) <5% U_T (>95% dip in U_T for 5 s)	<5% U_T (>95% dip in U_T for 0.5 cycle) 40% U_T (60% dip in U_T for 5 cycles) 70% U_T (30% dip in U_T for 25 cycles) <5% U_T (>95% dip in U_T for 5 s)	Mains power quality should be that of a typical commercial or hospital environment. If the user of the Ultimate Comfort requires continued operation during power mains interruptions, it is recommended that the WED-3000 B Ultrasound Scanner be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field according IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
REMARK U_T is the a. c. mains voltage prior to application of the test level.			

Guidance and manufacturer's declaration – electromagnetic immunity			
The Ultimate Comfort is intended for use in the electromagnetic environment specified below. The customer or the user of the Ultimate Comfort should assure that it is used in such an environment.			
Immunity tests	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
			Portable and mobile RF communications equipment should be used no closer to any part of the Ultimate Comfort, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.
			Recommended separation distance
Conducted RF according IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms	$d = \left[\frac{3,5}{V_1} \right] \sqrt{P}$
Radiated RF according IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	$d = \left[\frac{3,5}{E_1} \right] \sqrt{P}$ 80 MHz to 800 MHz $d = \left[\frac{7}{E_1} \right] \sqrt{P}$ 800 MHz to 2.5 GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m) ^b . Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, ^a should be less than the compliance level in each frequency range. ^b Interference may occur in the vicinity of equipment marked with the following symbol: 
REMARK 1 At 80 MHz and 800 MHz, the higher frequency range applies. REMARK 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			
a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Ultimate Comfort is used exceeds the applicable RF compliance level above, the Ultimate Comfort should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the Ultimate Comfort.			
b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.			

Recommended separation distances between portable and mobile RF communications equipment and the Ultimate Comfort			
The Ultimate Comfort is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Ultimate Comfort can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Ultimate Comfort as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output power of transmitter P in W	Separation distance according to frequency of transmitter l in m		
	150 KHz to 80 MHz $d = \left[\frac{3,5}{V_1} \right] \sqrt{P}$	80 MHz to 800 MHz $d = \left[\frac{3,5}{V_1} \right] \sqrt{P}$	800 MHz to 2.5 GHz $d = \left[\frac{7}{E_1} \right] \sqrt{P}$
0,01	0.12	0.12	0.23
0,1	0.38	0.38	0.73
1	1.20	1.20	2.30
10	3.80	3.80	7.30
100	12.00	12.00	23.00
For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.			
REMARK 1: At 80MHz and 800MHz, the separation distance for the higher frequency range applies. REMARK 2: These guidelines may not applicable in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

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Terms of Guarantee

You are now owner of a high quality Ritter product. Thank you very much for your trust in Ritter Concept GmbH and would like to maintain this! You have the warranty right of 12 months. We point to the warranty terms in our General Terms and Conditions.

We warrant, that our products are free of faults by the time of handover to the supplier. In particular, we warrant, that the production is taking place based on the acknowledged technical rules and the relevant regulations. We also warrant, that the product characteristics are available, which are determined in the contractual documents. In line with the warranty we perform a remedy of defects, when substances, parts or performances are identified as flawed or contrary to contract. The warranty starts with the date of delivery. Deficiencies are to be communicated with Ritter Concept GmbH immediately after receipt of goods and immediately after discovery. The customer is permitted to send back products without prior approval of Ritter Concept GmbH. The customer has to select the most economic way of transport.

No warranty claims exist in the following cases:

- using the product outside of or contrary to its intended use, or operating the product outside of its specifications
- failure to observe the instructions on use, maintenance or installation
- improper handling or neglected maintenance
- use of non-approved spare parts or accessories
- product modifications or conversions performed by customer or third parties
- usual wear and tear or other changes to or deterioration of the product attributable to:
 - external influences (e. g. knocks, blows, mechanical impacts)
 - water, fire
 - improper method of storage, handling or installation
 - unusual ambient conditions, especially conditions upon receiving or operating conditions at the installation location or force majeure
 - calcification or corrosion resulting from neglected maintenance or improper handling
 - contaminants in the air and water supply
 - unusual or impermissible chemical or electrical influences
 - soiling, discolouration or wear and tear on padding

The warranty as a rule excludes parts subject to normal wear, especially lamps, glass and rubber components and cables as well as plastic parts and surfaces regarding their colour-fastness.

All of our obligations under the warranty will be extinguished if the delivered goods are modified, improperly handled or processed. We do not warrant co-supplied or integrated third-party products. However, if requested we will assign to the customer our warranty claims against the vendor of the third-party product.

In order to activate the warranty, it is necessary to send a copy of the record of delivery and the warranty card to us following installation.